

CASSETTE

STAT⁺ CASSETTE



STATCASSETTE[®]

The STATCASSETTE[®] is a reliable and efficient urine cassette drug test device. See the back for information on test procedure and how easy it is to interpret these test results.

**CALL FOR A
FREE SAMPLE!**

TEST FEATURES

- FDA Approved
- Simple Dropper Method
- Minimal Volume Required To Run Test
- Results in 1-2 minutes
- Test Up to 15 Drugs (see below)
- Available with Built-in Specimen Validity Test
- Flat Face Allows for Photocopy of Results
- Preliminary Result Form Available

Drugs of Abuse:

Marijuana	Propoxyphene*
Cocaine	Fentanyl*
Opiates/Morphine	Tramadol*
Methamphetamine	K2/Spice*
Amphetamine	6-Monoacetylmorphine*
Phencyclidine	Cotinine*
Benzodiazepines	Alcohol*
Barbiturates	Ketamine*
Methadone	Bath Salts*
Ecstasy	Ethyl Glucuronide (ETG)*
Tricyclic Antidepressants	
Oxycodone	
Buprenorphine	
Methadone Metabolite	

*For Forensic Use Only



Toll Free: 866-933-0964
www.micro-distributing.com

STAT⁺ CASSETTE

EASY-TO-USE

FAST

ACCURATE



CALL FOR A FREE SAMPLE!

PROCEDURE


STEP 1

- Label the device with donor identification. Using the specimen dropper, withdraw the urine sample from the specimen collection container.



STEP 2

- Slowly dispense 3~4 drops (approximately 120µL) into the sample wells (S), being careful not to overfill the absorbent pad.

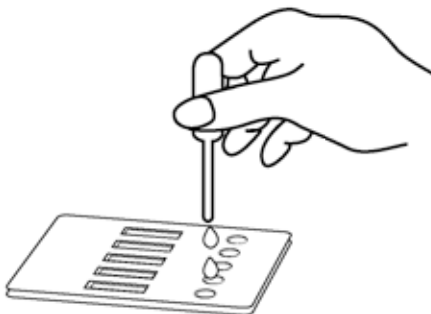

INTERPRETING RESULTS

- Read results at 5 minutes. Results are stable for 60 minutes.



NEGATIVE POSITIVE INVALID

Toll Free: 866-933-0964

Phone: 254-939-8923

Fax: 254-939-5867

sales@micro-distributing.com

