

STAT⁺ DIP



STATDIP[®]

The STATDIP[®] One-Step Dip Card is a simple and economical method to test for drugs of abuse. See the back for full procedure details and how easy it is to read results.

- Fast
- Accurate
- Reliable
- Easy-to-Use
- Affordable
- Dark Lines
- FDA Approved
- CLIA Waived
- Health Canada
- CE Marked
- ISO Certified Manufacturing

**CALL FOR A
FREE SAMPLE!**

TEST FEATURES

- Simple Dip & Read Procedure
- Available with Built-in Adulterants
- Dip Card Holds up to 14 Drugs
- "Kit" Packaging Available
(includes collection cups & gloves)

Drugs of Abuse:

Marijuana	Propoxyphene*
Cocaine	Fentanyl*
Opiates/Morphine	Tramadol*
Methamphetamine	K2/Spice*
Amphetamine	6-Monoacetylmorphine*
Phencyclidine	Cotinine*
Benzodiazepines	Alcohol*
Barbiturates	Ketamine*
Methadone	Bath Salts*
Ecstasy	Ethyl Glucuronide (ETG)*
Tricyclic Antidepressants	
Oxycodone	
Buprenorphine	
Methadone Metabolite	

Adulterants:

pH (acidic additives)
Specific Gravity (sample dilution)
Oxidants (bleach/oxidizing agents)

*For forensic use only. Not intended for treatment or diagnostic purposes.




Toll Free: 866-933-0964
www.micro-distributing.com

STAT⁺DIP

EASY-TO-USE FAST ACCURATE

CALL FOR A FREE SAMPLE!

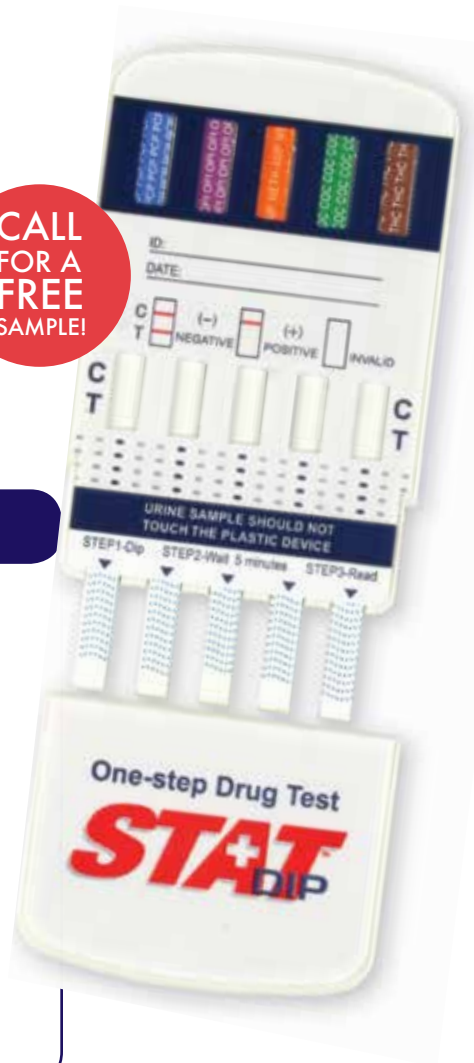

PROCEDURE

STEP 1

- Label the device with donor or control identifications. Remove the cap from the test device.



STEP 2

- Immerse the absorbent tip into the urine sample about 10-15 seconds until the migration of urine fluid is observed in the result windows. Urine sample should not touch the plastic device.




STEP 3

- Read results at 5 minutes. Results are stable for 60 minutes.



INTERPRETING RESULTS

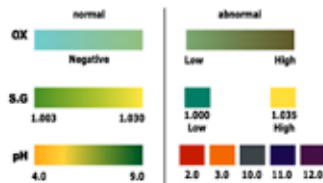

Interpret adulterant test strip between 3-5 minutes. Refer to enclosed color chart for an accurate interpretation.

Toll Free: 866-933-0964

Phone: 254-939-8923

Fax: 254-939-5867

sales@micro-distributing.com



www.micro-distributing.com