

ORAL FLUID TEST

STAT⁺SWAB



STATSWAB[®]

The STATSWAB[®] is a NEW and INNOVATIVE, fully integrated oral fluid drug screen device. The STATSWAB[®] is the simplest and fastest oral fluid drug screen on the market. Simply swab the mouth and screw the cap on. It's that easy! See back for details.

- Fast
- Accurate
- Reliable
- Easy-to-Use
- Affordable
- Dark Lines

CALL FOR A FREE SAMPLE!

TEST FEATURES

- Fully Integrated
- Simple Mouth Swab
- No Steps to Activate
- Observable
- No Restroom Needed
- Test Up to 14 Drugs
- Transport Sample to Lab

TEST AVAILABLE

Drugs of Abuse:

Marijuana	Methadone
Cocaine	Buprenorphine
Opiates	Ecstasy
Phencyclidine	K2/Spice
Methamphetamine	Cotinine
Amphetamine	Propoxyphene
Benzodiazepines	Alcohol
Oxycodone	
Barbiturates	

The STATSWAB is intended for forensic use, employment, and insurance testing. It is not intended for treatment or diagnostic purposes.

MD⁺

MICRO-DISTRIBUTING

Toll Free: 866-933-0964

www.micro-distributing.com

STAT⁺ SWAB

EASY-TO-USE

FAST

ACCURATE

CALL FOR A FREE SAMPLE!




NEW!

STAT⁺ SWAB PLUS⁺

Interpreting Alcohol Test Results

1. Read Results at 2 Minutes
2. Compare Reactive Pad to Color Blocks Below:
3. Interpretation:
 NEGATIVE - No color changes on reactive pad.
 POSITIVE - Reactive pad changes color. Estimate the approximate blood alcohol level by comparing the color of the reactive pad to the color chart below.

Approximate Blood Alcohol Level

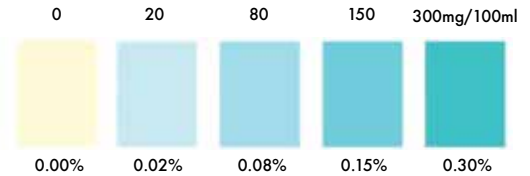

PROCEDURE

- Do not place anything in the mouth including food, drink, gum, or tobacco products for at least 10 minutes prior to collection of oral fluid specimen.


STEP 1

- Unscrew the Collector Cap to pull out the collection stick with Sponge from the Collection Chamber.



STEP 2

- Insert the sponge end of the collection stick into the mouth. Close mouth and actively move the sponge back and forth for saliva excretion. Gently pressing the sponge between the teeth and the tongue will assist in the sponge saturation process.
- Soak sponge into saliva in mouth and under the tongue for a total of 3 minutes until the sponge becomes completely saturated with saliva. No hard spots should be felt on the sponge when saturated.

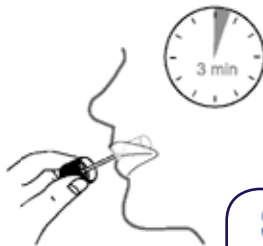


STEP 3

- Remove the sponge from the mouth. Place the collection stick with saturated sponge into Collection Chamber.



INTERPRETING RESULTS

- Peel off the label to read test results. Read results at 10 minutes. Do not read results after 1 hour.

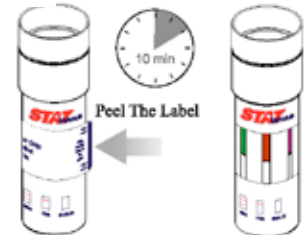

Toll Free: 866-933-0964

STEP 4

- Secure the cap and start the timer.



Phone: 254-939-8923
 Fax: 254-939-5867
sales@micro-distributing.com

MD[®]
 MICRO-DISTRIBUTING