


Intect® 7 Instructions

Instructions:

Dip the test strip into the urine collection cup and remove immediately. Be sure all pads are wet.




Blot the test strip gently on its side to remove excess urine. Results are ready in one minute.

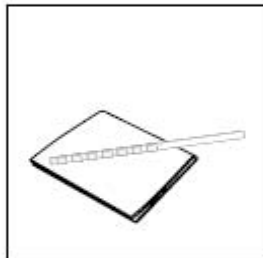

Results Interpretation: Compare the color of the test with the color chart and interpret. Results are void after 2 minutes.



Color Chart:

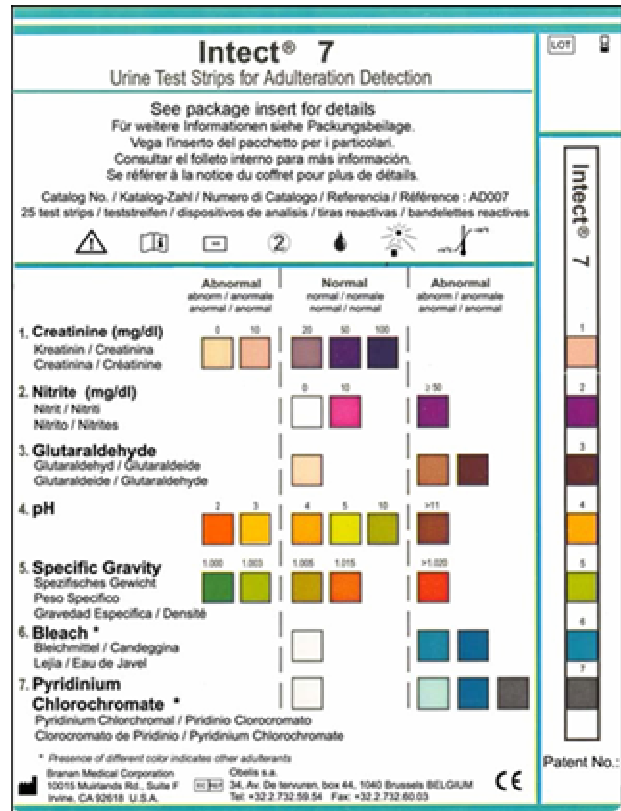

Use color chart on bottle for most accurate color comparison

Reactions:

	Creatinine	Nitrite	Glutaraldehyde	pH	Specific Gravity	Bleach	PCC
Classic Adulterants							
Bleach		•			•	•	•
Drano					•	•	•
Vinegar					•		
Glutaraldehyde			•		•		
Sodium Nitrite		•			•	•	•
Commercial Adulterants							
PCC		•			•	•	•
Urine Luck 6.3							•
Stealth						•	•
Clean-X						•	•
Klear		•					
Instant Clean		•				•	•
Purafyzit						•	•
UR'n Kleen			•				
Dilution							
Urine Dilution	•				•		

Test Features:

Accurate, most comprehensive test available
Detects adulteration, dilution and substitution
Simple to use and interpret
Results in 1 minute
No instrumentation required
Conforms to DOT guidelines
18 month shelf life
IntectCheck™ Controls available for monitoring performance