



Multi-Drug Screen Test Cup Procedure Card

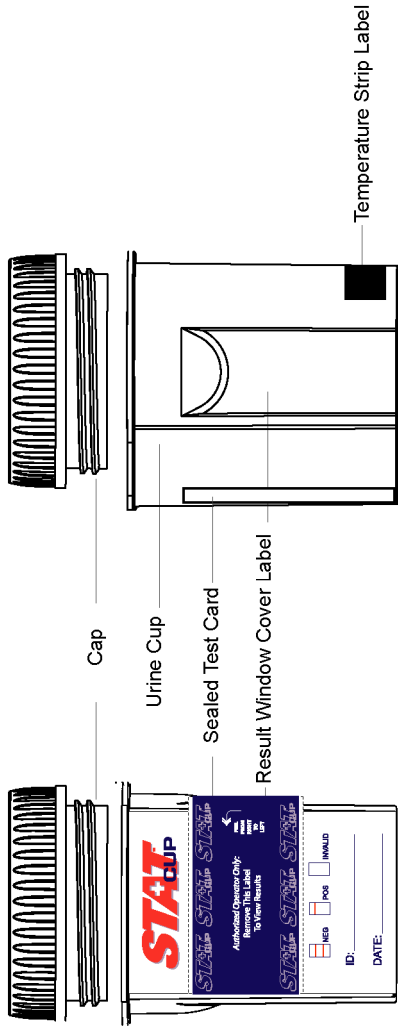

- 1** Remove Cap.
- 2** Donor provides specimen.
- 3** Replace the Cap securely. Peel off the blue label that covers the result window.
- 4** Read results at 5 minutes. Test results remain stable for a minimum of 60 minutes.

Negative results can be read as soon as they appear.
Read positive results at 5 minutes.

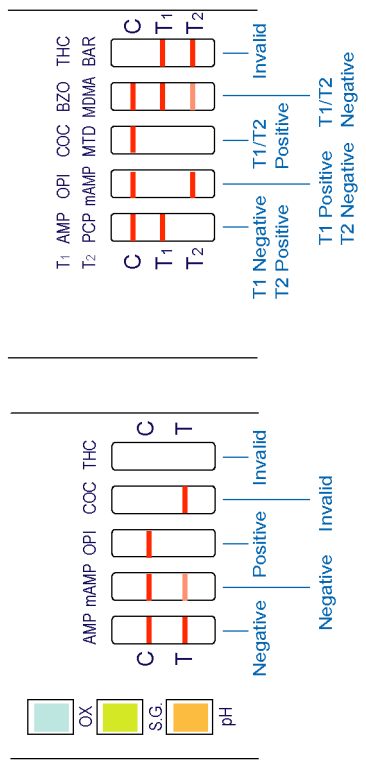

All positive results are presumptive and should be confirmed by an alternative method (e.g. GC/MS). A photocopy of the results can be made by placing the entire sealed cup with the card face down on the copier.

Interpret adulterant test strip between 3-5 minutes. Refer to enclosed color chart for an accurate interpretation.