


# Oral Fluid Drug Screen Device

# Procedure Card

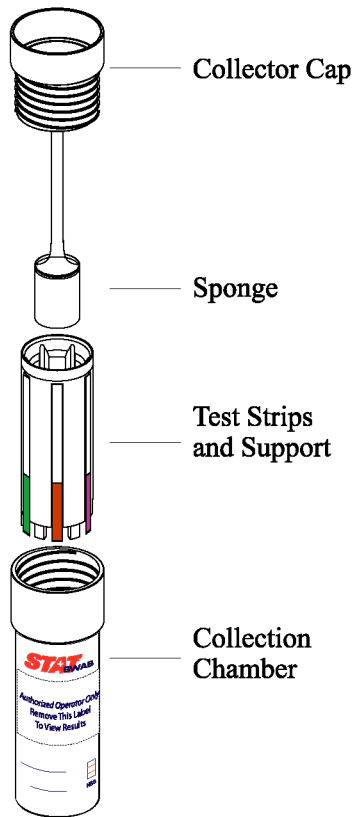

- Do not place anything in the mouth including food, drink, gum, or tobacco products for at least 10 minutes prior to collection of oral fluid specimen.

### STEP 1



- Unscrew the Collector Cap to pull out the collection stick with Sponge from the Collection Chamber.

### STEP 2

- Insert the sponge end of the collection stick into the mouth. Close mouth and gently chew the sponge for saliva excretion.
- Soak sponge into saliva in mouth and swab the inside of the mouth and tongue to collect oral fluid for a total of 3 minutes until the sponge becomes completely soft and fully saturated with saliva. No hard spots should be felt on the sponge when saturated.


### STEP 3

- Remove the sponge from the mouth. Place the collection stick with saturated sponge into Collection Chamber.

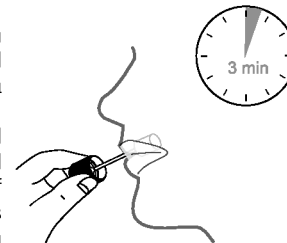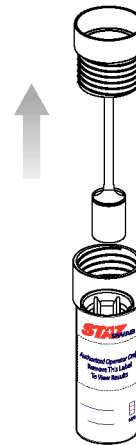

### STEP 4

- Secure the cap and start the timer.

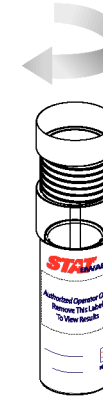

### STEP 5

- Peel off the label to read test results. Read results at 10 minutes. Do not read results after 1 hour.

